





Model SA-40 Quick Start Guide



HearAll

FOR SERVICE/REPAIR
CONTACT: TELTEX
(888) 515-8120 Voice
(866) 515-6316 TTY

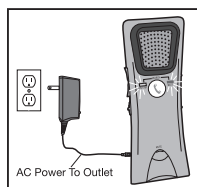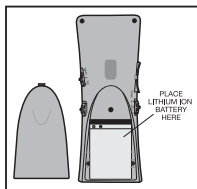

Charge the SA-40


1. Install the SA-40 Battery Pack.
2. Charge the SA-40 for 4 hours.

Pair the SA-40 with your Cellphone:

1. To perform the initial pairing, from your cellphone turn Bluetooth® "ON".
2. After charging for at least 4 hours, turn on the SA-40. The SA-40 will blink as it pairs to your cellphone. Follow the prompts on your cellphone to complete pairing. If your cellphone asks for a Pin Number, enter: 0000.

Select an Operating Mode:

1. **HS** - Use the SA-40 as a handset.
2. **T** - Couple with your T-Coil Hearing Aid.
Caution: If **accidentally** switched to the "**T**" position the SA-40 speaker will not work.
3. **SPKR** - Use as a speakerphone.

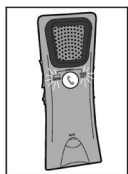

Need Help? Call Teltex Toll Free.

(888) 515-8120 (8:00am - 5:00pm CST M-F)

Answer A Call Using the SA-40:

1. When your cellphone rings, press to answer.
2. Adjust volume by pressing the **VOL (+)** or **VOL (-)** buttons.
3. To enhance clarity, move the **TONE** switch to the setting where you hear best.
4. Press to end the call.

Make A Call Using the SA-40:

1. Press to start.
2. If your cellphone is equipped with voicemail, speak the desired name or number into the SA-40.
3. If your cellphone does not have voicemail, enter the desired number on your cellphone keypad.
4. Press to hang up.

Please Note:

The SA-40 amplifies your callers voice up to 100 times louder than your cellphone. Use care when setting phone volume. The SA-40 should always be set to (HS) Handset Mode when handset is pressed to the ear.



800-841-6167
www.itactty.org

3001 Montvale Drive | Suite A
Springfield, Illinois 62704





Model SA-40 Quick Start Guide



HearAll

FOR SERVICE/REPAIR
CONTACT: TELTEX
(888) 515-8120 Voice
(866) 515-6316 TTY

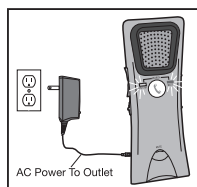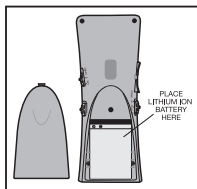

Charge the SA-40


1. Install the SA-40 Battery Pack.
2. Charge the SA-40 for 4 hours.

Pair the SA-40 with your Cellphone:

1. To perform the initial pairing, from your cellphone turn Bluetooth® "ON".
2. After charging for at least 4 hours, turn on the SA-40. The SA-40 will blink as it pairs to your cellphone. Follow the prompts on your cellphone to complete pairing. If your cellphone asks for a Pin Number, enter: 0000.

Select an Operating Mode:

1. **HS** - Use the SA-40 as a handset.
2. **T** - Couple with your T-Coil Hearing Aid.
Caution: If **accidentally** switched to the "**T**" position the SA-40 speaker will not work.
3. **SPKR** - Use as a speakerphone.

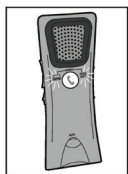

Need Help? Call Teltex Toll Free.

(888) 515-8120 (8:00am - 5:00pm CST M-F)

Answer A Call Using the SA-40:

1. When your cellphone rings, press to answer.
2. Adjust volume by pressing the **VOL (+)** or **VOL (-)** buttons.
3. To enhance clarity, move the **TONE** switch to the setting where you hear best.
4. Press to end the call.

Make A Call Using the SA-40:

1. Press to start.
2. If your cellphone is equipped with voicemail, speak the desired name or number into the SA-40.
3. If your cellphone does not have voicemail, enter the desired number on your cellphone keypad.
4. Press to hang up.

Please Note:

The SA-40 amplifies your callers voice up to 100 times louder than your cellphone. Use care when setting phone volume. The SA-40 should always be set to (HS) Handset Mode when handset is pressed to the ear.



800-841-6167
www.itactty.org

3001 Montvale Drive | Suite A
Springfield, Illinois 62704